













## CS 50/25 PRIME E

**Cod. PR-CS300-040-AN**


Oil-free compressor, 3 cylinders, 175 l/min @ 8 bar flow rate, 40 lt tank equipped with soundproof cabin and absorption dryer capable of delivering compressed air with filtration down to 0.01 micron and dew point below - 40 ° C at atmospheric pressure.

 Maximum simultaneous use: 3

 Dental Chairs: 4

										
Air Dryer	Air capacity	FAD @ 8bar	Motor	Voltage Hz	Receiver	Noise Level	Duty Cycle	max P.	Size	net Weight
Yes	m <sup>3</sup> /min 0.36 l/min 360	m <sup>3</sup> /min 0.175 l/min 175	kW 2.2	230V 50Hz	l 40	db(A) 53	100 %	bar 10	mm 720x490x970	Kg 103

## Air delivery performance

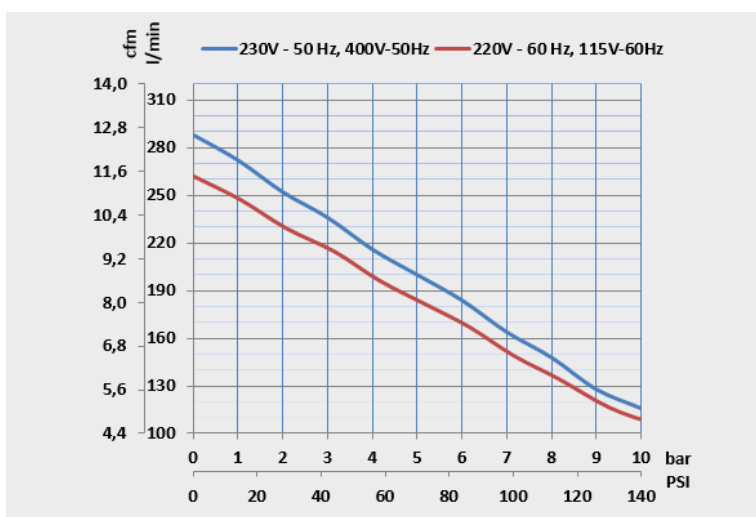

## Maintenance kits

Kit.	Description	Type	Hours
TA-FL-026-K-SCP	<p><b>PRIME 2-3 M filter cartridges kit</b></p> <p>Cartridge replacement kit for ordinary maintenance of the PURE AIR PRIME 2.3 M range.</p> <p>Comprehends:</p> <ul style="list-style-type: none"> <li>• 3 suction filters, code: TA-FL-058-C</li> <li>• 1 5 micron filter cartridge, code: TA-FL-001-C-SM</li> <li>• 1 0.01 micron filter cartridge, code: TA-FL-024-C2-SM</li> <li>• 2 o-rings for filters, cod. TA-FL-001-OR</li> <li>• 1 non-return valve pads, code: CO-PV-001</li> <li>• 1 5 micron filter cartridge (AF30), code: TA-FL-075-C</li> <li>• 1 0.01 micron filter cartridge (AFD30), code: TA-FL-076-C</li> <li>• 2 o-rings for filters, cod. TA-FL-075-OR</li> <li>• 1 AMG150 water separator filter cartridge, code: TA-FL-074-C</li> </ul> <p>Replace the filtration cartridges every 2 years.</p>	Ordinary	1.000h/1y
PR-GC-032-K-SCP	<p><b>PRIME 3</b></p> <p>Extraordinary maintenance spare parts kit for PRIME 2.2 kW (PRIME 3) versions.</p> <p>comprehends:</p> <ul style="list-style-type: none"> <li>• 3 suction filters, code: TA-FL-058-C</li> <li>• 6 O-rings, code: MA-PC018</li> <li>• 3 piston rings , code: MA-MG00001</li> <li>• 3 piston guide rings, code: MA-MG00003</li> <li>• 3 expansion rings for the segments, code: MA-MG00024</li> <li>• 3 complete valve plates, code: MA-MG-02002P-C-P</li> <li>• 1 non-return valve pads, code: CO-PV-001</li> </ul> <p>The recommended maintenance intervals for this compressor are 4000 hours/4 years.</p>	Extraordinary	4.000h/4y