



## MINI BOX 24/10 GENESI S










**Cod. D-MB150-024-15**

Oil free compressor, super silenced, FAD at 5 bar 135 l/min, tank capacity 24 L, without air dryer, complete with MINI BOX acoustic cabinet that can be installed also separately and in working environments. MINI BOX 2016 acoustic cabinet provides an optimal noise reduction (49 dB/1M) User friendly, compact and light, it can be placed everywhere.


 Maximum simultaneous use: 1

 Dental Chairs: 1-2

Medical Device CE 0425

 Air Dryer	 Air capacity	 FAD <small>Fad @ 5bar</small>	 Motor	 Voltage Hz	 Receiver	 Noise Level	 Duty Cycle	 max P.	 Size	 net Weight
No	m³/min    l/min 0.182    182	m³/min    l/min 0.135    135	kW 1.1	230V 50Hz	l 24	db(A) 49	50 %	bar 7	mm 530x500x800	Kg 67

## Air delivery performance

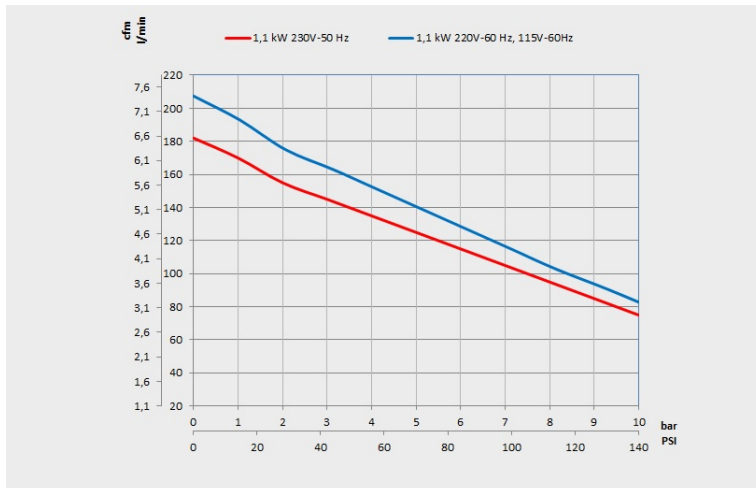

## Maintenance kits

Kit.	Description	Type	Hours
TA-FL-001-K	<b>GENESI S ordinary maintenance kit</b>  Ordinary maintenance kit for GENESI compressors, versions without dryer. The kit consists of the line filter cartridge (if present) and the pumping element suction filter. Annual replacement of kit components is recommended. Maintenance operations can be performed at a higher frequency if the environment in which the compressor works is particularly dirty.	Ordinary	1.000h/1y

Kit.	Description	Type	Hours
MG-GC-011-K	<p><b>Extra ordinary maintenance kit for GENESI S and M versions</b></p> <p>The GENESI extraordinary maintenance spare parts kit includes: a suction filter, two elastic rings, O-rings, a line filter cartridge and two complete valve plates. The recommended maintenance intervals for this compressor are 1500 hours / 2 years. Maintenance operations can be performed at a higher frequency if the environment in which the compressor works is particularly dirty.</p>	Extraordinary	1.500h/2y